

PORODO
LiveOn



Porodo Live On
Portable Nebulizer

SKU: PDN3WH

Table of Contents

Product Overview	2
Specifications	2
Before First Use	3
Product Components	4
Operation Instructions	5
battery Warnings	6
Disassembling	6
Adding Liquid	6
Assembling Method	7
Usage Instructions	7
Power Off	8
Cleaning and Maintenance	8
Disinfection	8
Safety Instructions	9
Package Contents	10
Disposal	10
Warranty	11
Contact Us	11

Product Overview

The Porodo LiveOn Portable Nebulizer is a rechargeable, compact, and efficient device designed to deliver fine particles for effective medication delivery. Its adjustable fogging feature allows you to customize the mist intensity for a comfortable experience, while the anti-leak and anti-backflow design ensures safe and reliable use. This nebulizer is perfect for medical-grade applications and operates quietly, making it ideal for both home and on-the-go use. With its detachable medicine cup, you can easily fill the container with the required liquid and assemble it with ease. The device offers flexibility with a mouthpiece or face mask mount, and the atomization starts with a simple operation, ensuring an effective, noise-free treatment.


Specifications

Battery Capacity	400mAh
Rated Voltage	3.7V
Nominal Frequency	108kHz
Rated Current	300m
Particle Size (MMAD)	4um
Working Time	60 Minutes
Charging Time	90 Minutes
Atomization Rate	≥0.2mL/min

Residual Liquid Capacity	≤0.5mL
Noise Level	≤ 50dB(A)
Operating Temperature	5°to 40°C
Pressure Range	86kPa to 106kPa
Water Resistance Level	IPX3
Product Weight	96g (Without Battery)
Product Size	61x47x125mm
Model Number	PDN3

Before First Use

1. When the storage temperature is lower than 5°C, the equipment should be returned to normal operating conditions before use.
2. Before installing and testing the product, please check the appearance of the product and the quantity of the accessories to be consistent with the list inside the manual.

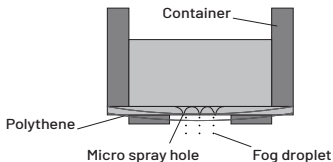


Product Components



Operation Instructions



1. Inject liquid into the medicine container.

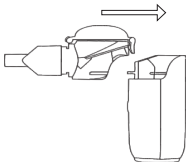


2. Assemble the medicine container.



3. Mount the face mask or mouthpiece.



4. Start atomizing the medication.

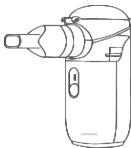

5. Use the mask or mouthpiece for treatment.

battery Warnings


1. Do not use incompatible batteries.
2. When the indicator light is red, it means the battery is depleted and the device will automatically switch off after 5 times of continuous blinking.
3. Do not insert the battery inside the machine when you don't use the nebulizer.

Disassembling

Press the PUSH button on the back of the machine, push the medication cup toward the front of the machine, and remove the cup.


Notes:

1. Ensure that you have pressed the PUSH button when removing the cup to avoid damaging the machine.
2. Do not insert anything into the spray outlet or touch the spray atomizer to avoid damaging the machine.

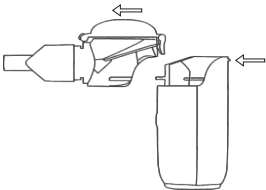


Adding Liquid

1. Do not use incompatible batteries.
2. When the indicator light is red, it means the battery is depleted and the device will automatically switch off after 5 times of continuous blinking.
3. Do not insert the battery inside the machine when you don't use the nebulizer.

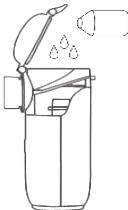


Assembling Method

You will hear a slight "beep" sound, indicating that the buckle has been secured when reassembling the machine. Then, ensure that the electrode is in good contact and can spray normally.

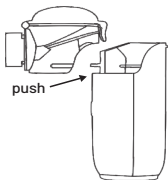

Usage Instructions

1. The first press of the power button will start the nebulizer with a strong spray. The second press will provide a normal spray, the third press will result in a light spray, and the fourth press will stop the nebulization.
2. When the liquid is about to run out, it is recommended that the user tilt the front of the nebulizer slightly toward the machine to facilitate the atomization of the remaining liquid that is in contact with the spray.
3. After pressing the switch, the nebulizer will go through a short start-up period within 2 seconds.
4. When using the mask for inhalation, do not cover the venting holes with your hands or other objects.
5. When holding the atomizing inhalation, hold the nebulizer smoothly by hand. Do not cover the venting hole on the lid of the cup with your hands or other objects, as this may affect normal spraying.
6. If the user feels any discomfort during use, please turn off the machine immediately and consult a doctor promptly.

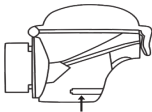

The liquid may remain here

Power Off

If you need to stop using the device during operation, turn off the power by pressing the start switch. The blue nebulizer indicator will turn off.

Cleaning and Maintenance

1. Remove the mouthpiece or mask from the cup, open the cup lid, and drain any remaining liquid. Add a small amount (2–5 mL) of pure water to the cup and close the lid. Gently shake the cup to dissolve any residue.
2. Open the lid and pour out the solution, then refill the cup with a small amount (2–5 mL) of purified water.
3. Press and hold the start button until the indicator light flashes blue. The nebulizer will enter cleaning mode, during which the purified water will be atomized to clean any remaining residue from the spray outlet.
4. For the main unit, use a damp gauze to gently wipe away stains, then dry it with a clean gauze.
5. Finally, clean the electrodes on both the main unit and the cup to ensure proper connection between the cup and the device.

Disinfection

- a. Be sure to disinfect the inhalation components (mouthpiece and mask) after each use. If any parts become heavily contaminated, replace them promptly. There are two recommended disinfection methods:
 - b. Alcohol disinfection: Disinfect the inhalation components using 75% medical alcohol.
 - c. Acetic acid solution disinfection: Disinfect the inhalation components by mixing one part white vinegar with three equal parts pure water to create a mild acetic acid solution.

Notes:

1. Do not wipe the mesh nebulizer with volatile liquids such as benzene, gasoline, or paint thinner.

2. Do not use cotton swabs or other objects to touch the electrodes on the main unit or the medication cup module, as this may cause them to loosen or fall off.
3. Remove the battery before cleaning any parts.
4. Always ensure the battery has been removed before cleaning.
5. The medicine cup is a consumable part and is not covered by warranty. Under normal conditions, its expected lifespan is approximately six months (based on usage of three times per day at normal room temperature or about 30 minutes per day). The actual lifespan may vary depending on the medication used.
6. If residual liquid is difficult to remove, rinse the cup with warm water or repeat the washing process as needed.
7. Clean the medicine cup after each use to prevent clogging of the atomizer spray outlet and to maintain proper atomization.
8. The cleaning mode is intended only for cleaning the atomizer after use and must not be used for normal liquid atomization.

Safety Instructions

1. After disinfection, rinse thoroughly with pure water to remove any alcohol or acetic acid residue.
2. Do not drop the main unit or the medicine cup module, and avoid subjecting them to strong impact or shock.
3. Do not insert needles or other sharp objects into the nebulizer.
4. Do not expose the main unit or other components to extreme heat, extreme cold, or direct sunlight.
5. Do not plug or unplug the power adapter with wet hands.
6. This product should be purchased and used under the guidance of a doctor.
7. When storing or using the nebulizer, keep it out of reach of children or individuals with mental illnesses. Use is permitted only under adult supervision.
8. The nebulizer may be used by multiple users; however, accessories that come into direct contact with the body (such as masks and mouthpieces) must be assigned to a single user to prevent cross-infection. If multiple users share the main unit, each user's accessories should be stored separately.
9. Do not disassemble the main unit or attempt to repair it yourself.
10. Do not store the nebulizer if medicine or water is remaining in the medicine cup.

Package Contents

Main unit x1

Medication cup module x1

Mouthpiece x1

Masks x2

Disposal

This product must not be disposed of as unsorted household waste. It is important to separate such waste for proper treatment and recycling, in compliance with local waste management regulations.



Warranty

Products that you buy directly from our **Porodo** website or shop come with a 24-month warranty.

When you buy **Porodo** products from any of our approved sellers, you only get a 12-month warranty. If you want to extend this warranty, go to our website at **<https://www.porodo.net/warranty>** and fill out the form with your information. Don't forget to upload a picture of the product too. After we've checked and accepted your request, we'll send you an email to confirm that your product's warranty has been extended.

For more info, please check:
<https://www.porodo.net/warranty>

Contact Us

If you have any questions about this Privacy Policy, please contact us at:
info@porodo.net

Website: **<https://www.porodo.net/>**

Service Support: **support@porodo.net**

Instagram: **[porodo](#)**